THUASNE

Rollator Quatro

Mode d'emploi

User manual

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Manuale

Manual

Français	3
English	13
Deutsch	23
Nederlands	33
Italiano	43
Espanol	53

Français

Table des matières

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Safety instructions

Pay attention on this sign. It serves your security.

- On the rollator this sign means a hint for passages in the user's manual, that have to be read carefully and obeyed strictly.
- Passages marked with this sign in the user's manual have particular significance.

Please consider every time the hints in this user's manual and keep it carefully. Improper use of the rollator or use not in accordance with this manual may cause danger or damage to the rollator.

Only use the rollator when he is folded out and with locked cross tube. Check that the folding lock is safely engaged.

Control daily that the folding lock is free of dust or dirt.

If you want to use the seat of the rollator, you have to fix the parking brake before you sit down. When you sit down, you have to look in the opposite of your former walking direction. Always sit upright on the seat. Do not lean to one side or sit oblique on the rollator seat. The seat is created as an aid for a short rest and has not the function of a chair.

The hand grips shall not be adjusted turned outside. They must be parallel to the walking direction.

Please check before use if the clamping levers are well tightened and point down vertically.

Use the rollator only on solid and even ground. Do not ride over grids or drain holes, because the tyres could stick in trenches or grooves.

Load both grips equally. Use both brakes equally. Especially when you ride backwards, you should brake very carefully.

On dirty, wet, icy or snowy ground there is a risk of slipping and a decrease of brake power.

Do not overload the rollator. The maximum weight of additional load is 10 kg.

When the rollators is not in use, it is advisable to lock the brakes.

Introduction

By purchasing the rollator **Quatro Marathon** or **Quatro Exclusive** you have chosen a very stable rollator which will help you to move inside and outside of the house. Now you can make your shopping or shorter walks independently.

More detailed it can be used

- Indoor:
e.g. in a rehabilitation centre or nursing home, with sufficient room for manoeuvrability, no doorsills or uneven surfaces.
- Outdoor and indoor
Wherever you have lack of room for manoeuvrability, doorsills, steps / footboards, uneven surfaces, e.g. in an unadapted dwelling, outside on the street, in shops or on public transport.

The rollator is delivered with a removable shopping basket, a small luggage rack and a tray. It is equipped with a comfortable seat so that you can rest during a walk. It can be folded up easily and you can take it everywhere.

If you do not need your rollator place it with locked brakes in a manner that no one can fall over it (e.g. a corner of the room).

If you do not use your rollator for a longer time fold it as described on page 18. Fix the handles in the lowest position. You should also protect it against dust by using the carton or a plastic bag.

Do not store the rollator outside to avoid damages by high humidity, dirt and direct sunlight over long time.

If you want to carry your rollator with you (e.g. in a car) fold it as described on page 18. If necessary you can push the handles completely in (see "Assembly", page 16). To carry the rollator safely you shall grip it at the cross tube.

Delivery

Check by delivery if the product complies with the ordered rollator. Inform your dealer if there are deviations between the delivered and ordered rollator.

Your dealer hands the rollator to you in an usable condition. Before the delivery it will be adjusted to the correct height for you.
Your dealer should explain the use and the different functions to you.
Check by yourself also that the rollator works correctly.

Transport damage

Should your rollator be damaged during the transport, inform your dealer about this fact immediately.

Overview

1. Seat
2. Hand brake
3. Fixation screw for height adjustment of the handgrips
4. Cross tube
5. Rear wheel with brake independent from tire pressure
6. Castor wheel
7. Shopping basket
8. Tray
9. Stick holder
10. Folding lock

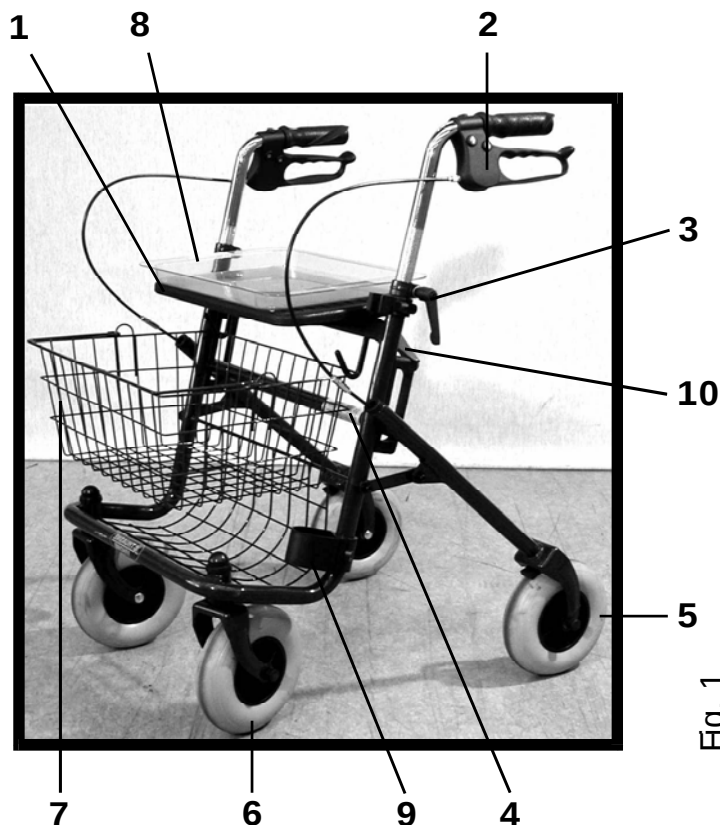
Fig. 1

Assembly

When delivered in packing material the Quatro Marathon rollator should be assembled in the following way:

1. Remove the rubber plugs and plastic bag;
2. If there is a horizontal tube between the back wheels, you have to remove it. It serves transport issues.
3. Put the wheels on the ground
4. Pull the under frame in your direction and push the cross tube (Fig. 1) down.
5. Stick the grip tubes in the frame.

Each grip tube must be stuck minimum 10 cm in the frame. The maximum height adjustment is marked on the grip tubes with the words „MAX.“ or „Security“ and has to be kept. (Fig. 2)

The hand grips shall not be adjusted turned outside. They must be parallel to the walking direction.

Fig. 2

6. Tight the fixation screws firmly.

Make sure that the fixation screws point down to avoid that you get caught on something.

If the position of the fixation screw is unfavourable, pull the levers outside and turn them in the required position. The functionality is not affected thereby.

Only model Quatro EXCLUSIVE

(Fig. 3)

7. Hang the braking cable holder on the hook under the seat.

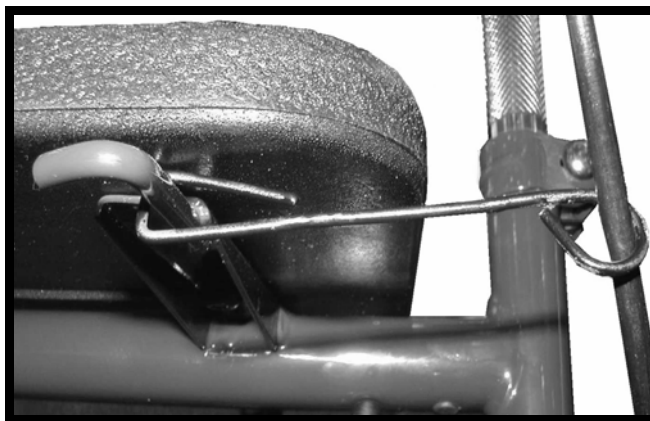
Fig. 3

Folding out

Pull the under frame of the rollator in your direction and push the cross tube completely down until it engages in the plastic guide.

Quatro Marathon

Fold down the red lever entirely to lock the cross tube safely.

Quatro Exclusive

This model has an automatic fold lock. With folding out the rollator the red lever falls down automatically and locks the folding process.

Fig. 4

The rollator is ready for use now. You can put up the basket and/or the tray.

Tray

Small objects can be put on the tray. On the bottom of the tray are four small feet, which have to be engaged to the holes in the seat (Fig. 5).

The tray is not heat resistant.
Transport of heavy or sharp-edged objects has to be avoided.

The maximum adjustable weight is 5 kg.
Load it proportionate.

Fig. 5

Shopping basket

The shopping basket is attached on two hooks under the rollator seat. (Fig. 6).

The shopping basket is only suitable for the carriage of objects. It must not be overloaded.

The maximum adjustable weight is 5 kg.

Fig. 6

Folding in

Remove the basket and the tray. Loose the brakes, turn the red lever to the side and pull the cross tube upwards.

On the model Quatro Exclusive you have to hold the folding lock.

Hand grips

Turn the fixation against the clockwise (Fig. 7).
Adjust the hand grip to the required height and turn the lever clockwise to fasten the grip again. Adjust the second grip at the same height as the first.

Fig. 7

Pay attention that the levers point downwards after all adjustments. If the position of the fixation screw is unfavourable, pull the lever outside and turn them in the required position. The functionality is not affected thereby.

Each grip tube must be stuck minimum 10 cm in the frame. The maximum height adjustment is marked on the grip tubes with the word "MAX." or "SECURITY" and has to be kept (Fig. 8).

The hand grips shall not be adjusted turned outside. They must be parallel to the walking direction.

"SECURITY"
" or "MAX."

Fig. 8

Brakes

The rollators Quatro can be equipped with different brake types.

Quick brake

- To use the brake function, pull up the brake levers slowly (Fig. 9)
- Fixing the brakes: Push the levers down beyond the perceptible pressure point.
- Taking off the brake: Pull up the levers, until the brake is unlocked.

Fig. 9

Scissor brake

- To use the brake function, pull up the black brake levers slowly (Fig. 10)
- Fixing the brakes: Pull up the red levers to the handgrips beyond the perceptible snap point.
- Taking off the brake: Pull up the black levers.

Fig. 10

Seat

If you want to use the seat, please take care of the following items before sitting down.

The rollators has to be placed on even solid surface.

If you want to use the seat of the rollator, you have to fix the parking brake before you sit down. When you sit down, you have to look in the opposite of your former walking direction. Always sit upright on the seat. Do not lean to one side or sit oblique on the rollator seat. The seat is created as an aid for a short rest and has not the function of a chair.

Maintenance

A damaged rollator must not be used before professional repairing.

The rollators can be cleaned easily by hand with a mild soapsuds or a usual mild detergent. Afterwards it has to be washed with pure water and dried with a soft cloth. Don't use a high pressure cleaning machine, because it may damage the bearings. A regular maintenance of the rollator is not necessary if it is used in accordance to the hints in this manual. Before every use check its functions, especially the brakes.

Brakes:

Wear can impair the function of the brakes. Please check the faultless function of the brakes before every use. Please check also the brake cable for deformation or pricks. The brakes can be adjusted. Works on this important security element must be done by professional technicians. Your specialized dealer can adjust them exactly to your wishes.

Tires:

Strong worn tires or tires with pricks have to be exchanged. Can all wheels turn freely? Please, check weekly.

Fold lock:

Check daily the function of the fold lock.

Model Quatro Exclusive:

Does the fold lock bolt automatically? Does the fold lock in locking position by itself when folding out?

Usually the mechanism doesn't need any maintenance. Only strong soiling or fluffs have to be removed.

Frame

Check the frame regularly for deformation or pricks.

Technical data

Length	: 67,5 cm
Width	: 59,5 cm
Dimensions	: 88x59,5x26,5 cm
Adjustable height	: 78,5 – 98 cm
Tyre size	: 200 x 50 mm (Quatro Marathon) 200 x 45 mm (Quatro Exclusive)
Maximum weight (Transport)	: 11,1 kg
Maximum user weight	: 130 kg
Maximum loading of the racks	
Basket	: 5 kg
Tray	: 5 kg
Luggage rack	: 10 kg
Maximum loading on the racks altogether	: 10 kg
Total weight	: 12,2 kg
Max. ankle adjustment of the grips	: 15°

Warranty

We take over the warranty according to the European lawful regulations. The first 12 month we take over guarantee. The guarantee includes mistakes of production and assembling. We are not liable for tear and wear, careless and/or improper use.



The warranty expires in case of repairing or changing the product done by not authorized dealers or persons. It does not include costs for repairing mistakes, for carriage, for loss of utilization and earnings, for necessary approaches or similar costs.

We cannot be held liable for damage or injury resulting from improper use, abuse or disregard of this user manual.

Technical changes prohibited.

Your specialised dealer:

Spare Parts

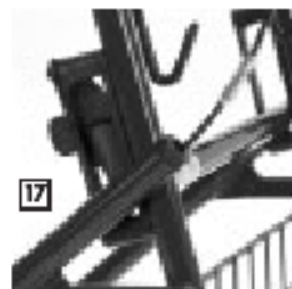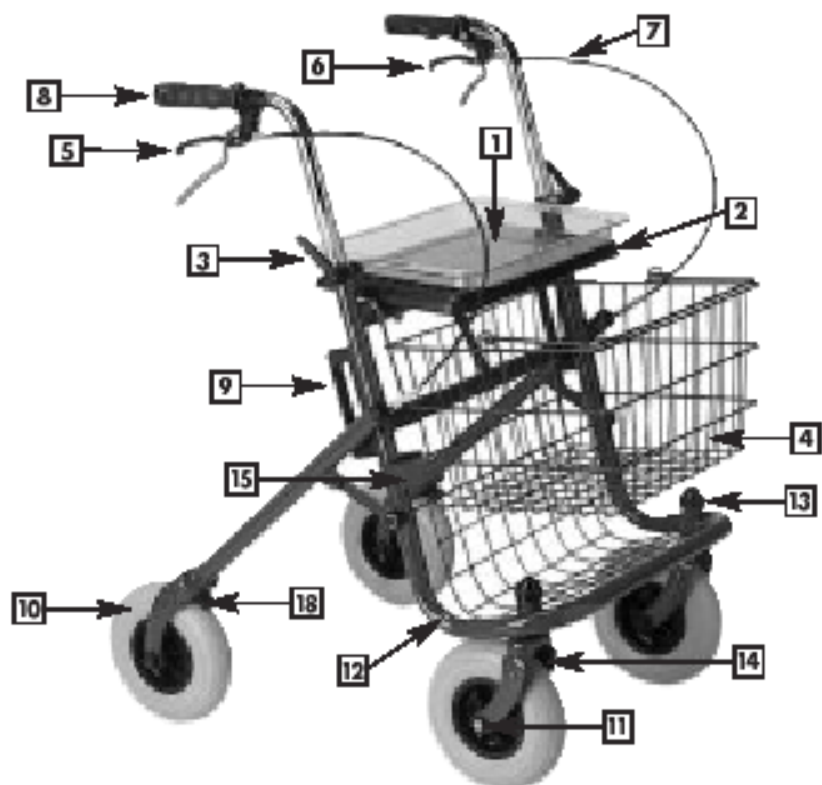	Description	Art.-no.
1	Tray	V0520200
2	Seat (22 x 38 cm)	V0520202
3	Clamp screw	V0520203
4	Basket	V0520204
5	Grip for scissor brake, right	V0520222
6	Grip for scissor brake, left	V0520223
7	Bended cable for brake	V0520205
no. pic.	Cable holder (only Quatro Exclusive)	V0520245
8	Anatomic grips (pair)	V0520207
9	Plastic bracket for folding lock	V0520208
10	TPU-wheel, complete (for Quatro Exclusive)	V0520209
no. pic.	PU-wheel (for Quatro Exclusive)	V0520249
11	Bearing front fork	V0520210
13	Dust cap for front fork	V0520211
14	Front fork, complete	V0520212
15	Cane holder	V0520050
16	ERGO-brake, left/right	V0520220
17	Folding lock (Quatro Marathon)	V0520213
no. Pic.	Folding lock (Quatro Exclusive)	V0520243
18	Brake shoe	V0520246
	Brake shoe set complete	V0520206

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MEDICAL DEVICES DIRECTIVE 93/42/EEC

- Declaration of conformity - for Class 1 Medical Devices

Manufacturers name: Thuasne Nederland BV

Manufacturers address: Gildenstraat 15
3861 RG Nijkerk
The Netherlands

Product: Rollator

Product name: Quatro Marathon
Quatro Exclusive

I, the undersigned, hereby declare that the equipment specified above, is compliant with the Essential Requirements of Medical Devices Directive 93/42/EEC. The product is signed with the CE-mark according to the above directive.

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THUASNE NEDERLAND B.V.